



DYNAMIC
DENTAL EQUIPMENT

REPLACING DYNAMIC DISC GUIDE FOR HORSEPOWER HANDPIECES


YOU WILL NEED:

AVAILABLE FROM YOUR DYNAMIC DISTRIBUTOR:

- A) DYNAMIC DISC REMOVAL TOOL
- B) HORSEPOWER HEAD LOCKING TOOL
- NEW DYNAMIC REPLACEMENT DISC

YOU WILL ALSO NEED:

- AN ADJUSTABLE SPANNER
- LOCTITE NUMBER 243

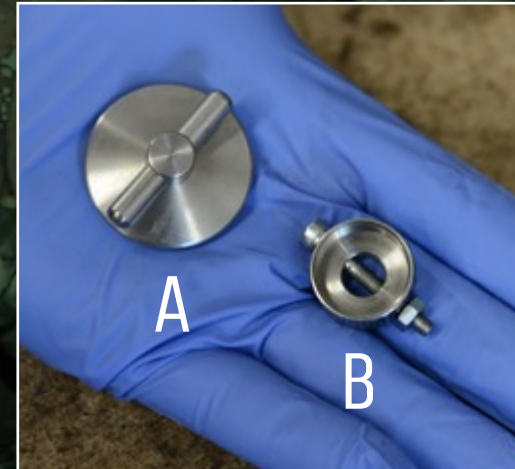



- REMOVE HEAD FROM HAND PIECE
- HEAT THE HORSEPOWER ADAPTOR WITH A HEAT GUN AROUND THE JOINT (HIGHLIGHTED)



- REMOVE THE HORSEPOWER ADAPTOR FROM THE BASE OF THE HEAD





▪ SCREW THE HORSEPOWER SPECIFIC HEAD LOCKING TOOL ON TO THE BACK OF THE HEAD CASING



▪ MAKE SURE THE HOLES LINE UP AND THEN THREAD THE SCREW THROUGH THE LOCKING TOOL AND HEAD, ATTACH THE NUT TO MAKE SURE IT IS TIGHT.





- PLACE THE DISC REMOVAL TOOL OVER THE DISC



- ENGAGE THE DISC REMOVAL TOOL OVER THE DISC AND ROTATE ANTI-CLOCKWISE



- THEN REMOVE THE OLD DISC AND DISCARD



- INSERT THE NEW DISC INTO THE DISC REMOVAL TOOL AND PLACE INTO THE GUARD



- TIGHTEN THE DISC IN A CLOCK-WISE ROTATION UNTIL SECURE.



- REMOVE LOCKING TOOL FROM HEAD AND REATTACH THE HORSEPOWER ADAPTOR TO THE BASE OF THE HEAD PLACE A SMALL AMOUNT OF LOCTITE ON HEAD CASING THREAD



- SECURE HEAD BACK ON TO THE HAND PIECE

